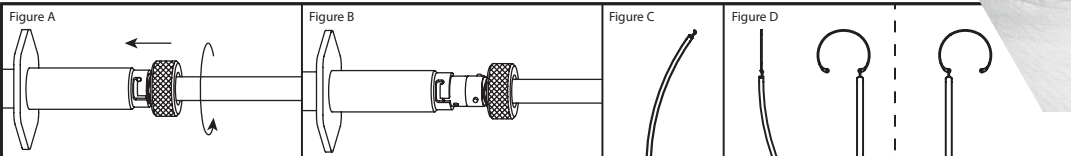
Duckworth & Kent Ltd

Titanium Reusable Ophthalmic Instrument Manufacturer

7-811

Capsule Tension Ring Inserter

Description:

The 7-811 Capsular Tension Ring Inserter is a reusable medical device, designed to insert Ophtec Capsular Tension Rings.

Indication for use:

The 7-811 Capsular Tension Ring Inserter is used to deliver the Ophtec Model 275 and Model 276 Capsular Tension Rings into the capsular bag.

First use:

The 7-811 Inserter is supplied "Non-sterile" and must be cleaned and sterilised before first use.

Cleaning & Disinfection:

The Capsular Tension Ring Inserter is comprised of 2 components that come apart for cleaning. By pushing and rotating the knurled ring, the body and plunger are separated. (g A)

Manual Cleaning:

Rinse excess soil from the device.

Fully immerse device into detergent solution not exceeding 30° C.

It is recommended that the device be cleaned as soon after use as possible, however where blood, tissue, saline or viscoelastic has been left to dry it is recommended that the device is left to soak for 30 minutes in the detergent solution. Using a brush, vigorously wash and scrub, applying detergent solution to all surfaces.

It is important to ensure that no air is trapped inside the device and that the detergent covers all surfaces.

After manual cleaning, rinse the device in clean water for a minimum of three times.

Disinfection:

Disinfectant solution may be used in accordance with label instructions of the disinfectant manufacturer. After manual disinfection, rinse the device with the freshly distilled or deionised water for a minimum of 3 times. Ensure that running water passes through the device.

Automated cleaning:

Place the 2 separated components in a suitable container and subject to an automatic dishwasher cycle, consisting of 3 key elements, a 2 minute pre-wash, a 3 minute detergent wash at 93° C and a drying stage sufficient to ensure complete absence of moisture on the 2 components.

Avoid the use of acid based products when cleaning the device and always follow the guidelines set by the detergent and mechanical cleaner manufacturer.

If possible, sterilising trays should be used to hold the device in a secure position during cleaning and storage to prevent damage. Store device in a dry area. The above instructions have been validated.

Sterilisation:

Moist heat (steam) in autoclavable bags is the preferred method of sterilisation. Only products that have been cleaned and disinfected can be sterilised.

Use either CE marked or validated vacuum autoclave, always following the instructions of the machine manufacturer. When sterilising multiple devices in one autoclave cycle, ensure that the steriliser's maximum load is not exceeded. Ensure devices are dry before sterilisation.

A sterilisation cycle of 134° C with a holding time of 3 minutes has been validated.

Reprocessing validation:

All cleaning and sterilisation processes require validation at point of use.

Re-assembly of the device:

To re-assemble the Inserter, gently push the plunger into the main body and align the pin with the slot in the main body. Engage the pin in the slot and rotate the plunger clockwise until the pin locks into place.

(g B)

Instructions for loading and implantation of Capsular Tension Ring

Carefully read all instructions for use before loading & implanting the Capsular Tension Ring.

Preparation of the Inserter

• Clean and sterilise the Inserter according to the cleaning and sterilisation instructions.

• Check the mechanism is working by fully compressing the plunger of the instrument to expose the grasping hook; make sure that the hook is free of debris and operates correctly (g. C).

Preparation of the Capsular Tension Ring

• Using sterile techniques, remove the container from its primary package;

• While holding the container upright, twist the cap to expose the Capsular Tension Ring;

• It may be necessary to rotate the Capsular Tension Ring within the container to provide optimum positioning of the eyelet;

• The Capsular Tension Ring is ready to be loaded into the Inserter.

Loading the Capsular Tension Ring

• Fully compress the plunger of the inserter to expose the grasping hook (g. D);

• Carefully insert the hook into one eyelet at the end of the Capsular Tension Ring;

• Slowly release the plunger to retract the Capsular Tension Ring into the barrel of the Inserter

• As soon as the hook enters the barrel, gently lift the Capsular Tension Ring from its container while completing the retraction;

• Make sure that the Capsular Tension Ring does not become entangled during retraction;

• When the Capsule Tension Ring is entirely drawn back into the tip of the inserter, then;

• The Capsular Tension Ring is ready for implantation. For best results it should be implanted immediately after loading. If the delivery is delayed, the haptic will take longer to return to its original configuration.

Implantation of the Capsular Tension Ring

See the instructions for use of the Capsular Tension Ring.

Cleaning

It is essential to clean the Inserter thoroughly immediately after use according to the Cleaning & Sterilisation Instructions.

Warnings and precautions

• The Capsular Tension Ring should be implanted immediately after loading;

• It is essential to clean the Inserter directly after implantation of the Capsular Tension Ring because intraocular fluids, irrigating solutions or viscoelastic substances may have entered the instrument and should be removed thoroughly.

Duckworth & Kent Ltd.
Terence House
7 Marquis Business Centre
Royston Road, Baldock,
Hertfordshire, SG7 6XL
United Kingdom

Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands

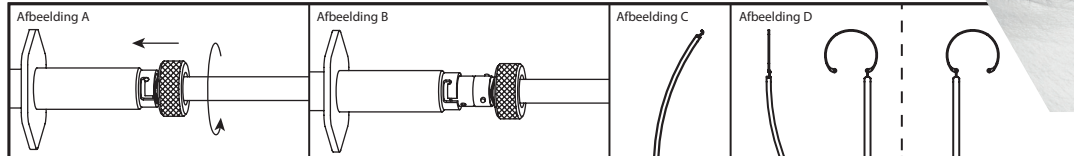
Duckworth & Kent Ltd

Titanium Reusable Ophthalmic Instrument Manufacturer

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