## ARTIFLEX Reusable Implantation Forceps Right

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Duckworth & Kent Ltd(영국, Terence House, 7 Marquis Business Centre, Royston Road, Baldock, Hertfordshire, SG7 6XL, UK)

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## ARTIFLEX Reusable Implantation Forceps Right

- Product name: Ophthalmic forceps, tissue, reusable
- Product license no.: Seoul Sushin 12-1023
- Model name: OF106

### [Intended Use]

It is reusable medical device used for grasping eye or peripheral tissues of the eyes in ophthalmic area.

### [Instructions for use]

#### A. Preparation before use

- 1) Check the device maintains clean and sterilization for use
- 2) Before use, check the device for deformed appearance, pinhole, foreign substance adhesion etc

#### B. Instruction for use

- 1) After selecting the patient, choose the tool that suits the purpose and function.
- 2) Grip the handle of ophthalmic forceps.
- 3) During surgical procedure, grasp the eye or peripheral tissues of the eyes or use it for ARTISAN/ARTIFLEX lenses insertion into the intraocular.

#### C. Storage and Care method after use

##### Re-sterilization

- 1) Moist heat(steam) in autoclavable bags is the preferred method of sterilization.
- 2) Only products that have been cleaned and disinfected can be sterilized.
- 3) When sterilizing multiple devices in one autoclave cycle ensure that the sterilizer's maximum load is not exceeded.
- 4) A sterilization cycle:
  - A temperature of 126°C with a holding time of 26 minutes
  - A temperature of 134°C with a holding time of 3-3.5 minutes

##### Storage

- 1) Ensure devices are dry before storage, and stored in dry, clean conditions at an ambient room temperature

##### Cleaning

- 1) Rinse excess soil from device.
- 2) Fully immerse device into a detergent solution not exceeding 30°C.
- 3) It is recommended that the device be cleaned as soon after use as possible, however where blood, tissue, saline or viscoelastic has been left to dry it is recommended that the device is left to soak for 30 minutes in the detergent solution.
- 4) Using a brush, wash and scrub vigorously applying detergent solution to all surfaces ensuring that hinged devices are cleaned in both open and closed positions.

##### Drying

- 1) When drying is achieved as part of a washer disinfector cycle, do not exceed 170°C.
- 2) Products may be dried using filtered compressed air.

##### Inspection

- 1) After cleaning, examine the device closely.

- 2) Check the lubrication of moving part and the operation status.
- 3) Check the cleaning status for effective sterilization.
- 4) Remove for repair or replacement any blunt, worn out, fractured or damaged devices.
- 5) If ANY soil or fluid is still visible, return the device for repeat decontamination.

### [Cautions in use]

#### A. Cautions in use

- 1) Follow instructions and warnings as issued by manufacturers of any decontaminants, disinfectants and cleaning agents used. Wherever possible avoid use of mineral acids and harsh, abrasive agents.
- 2) The products are precision surgical devices, the utmost care must be taken at all times when handling these devices to avoid damage.
- 3) No part of the process shall exceed 170°C.
- 4) When reprocessing medical devices, always handle with care, wearing protective clothing, gloves and eyewear in accordance with local Health & Safety procedures.
- 5) End of life is normally determined by wear and damage due to use.
- 6) Stains may occur when the device is sterilized, so follow the instructions from the sterilizer manufacturer, and do not open the cover until the steam is completely removed.
- 7) When water containing a lot of iron is used during cleaning, a film painted with rust may be formed, so use after discussing the rating evaluation of the water treatment problem with expert.
- 8) Corrosion can occur when blood is buried and harden at the deadlock, chopper and rings, so completely open up all deadlock before cleaning.
- 9) When exposed to salinity solution, blood, calcium chloride and many other compounds, the dent may occur, so rinse with distilled water as soon as possible after exposures.
- 10) Use only for the designated purpose and do not use it for other purpose.
- 11) If problems occur during storage and use, contact the manufacturer for follow-up and prevention of recurrence.

### [Storage method]

Store at room temperature

### [Manufacturer]

Duckworth & Kent Ltd(Terence House, 7 Marquis Business Centre, Royston Road, Baldock, Hertfordshire, SG7 6XL, UK)

### [Name and the address of the importer]

Ophtec Korea Ltd, 7FL, Teheranro 87Gil 33, Gangnam-gu, Seoul

**This product is a Medical Device**

Written Month/Year: November 2020 (PS-106 rev3.0)